

# STOCKINGAID

*Aids application of open toe compression and anti-embolism stockings*

The **StockingAid** is an inexpensive aid for putting on open toe compression and anti-embolism stockings. It is made from our proven SlideMate™ materials in both single patient use and washable. It rolls over on itself when removing it to minimise friction.

The StockingAid is suitable for use with open toe compression and anti-embolism stockings. It is important that anyone using the StockingAid is adequately trained in both its use as well as application of compression and stockings.

Ensure proper sizing of stockings and always refer to the manufacturer's instructions.

## STOCKINGAID SINGLE PATIENT USE

- Yellow
- Polyester strap on end
- Individually sealed with label and instruction
- Reduce risk of infection between patients
- Ensures one is available as required by patients rather than waiting for the laundry
- Often more cost effective considering wash costs and loss of reusables



Code: **BH-SASPU**

Qty: **BOX 50**

## STOCKINGAID PREMIUM

- Blue
- Polyester strap on end
- Individually sealed with label and instruction
- Research shows material washable to 250 washes



Code: **BH-SA** / Qty: **EACH**

Code: **BH-SA-25** / Qty: **BOX 25**

*For correct fitting and usage of your compression or anti-embolism stockings please discuss with your clinician. These stocking aid instructions are a guide only and do not replace the fitting guidelines by the stocking manufacturer. Before use, confirm with the manufacturer of the compression or anti-embolism stocking that the SallyStocking Aid is validated for use with their stocking.*



*To start ensure the StockingAid is folded in on itself so it is double layer*



*Insert StockingAid over the foot pulling it up over the heel*



*Pull open toe compression stocking over StockingAid until it is in place and strap can be easily pulled on. Ensure the StockingAid is not bunched up in any areas, especially under the knee.*



*Pull StockingAid handle away from client and slide out easily.*



*The compression stocking is now on and in place.*